

# Plastic-free Universal Wipes

clinell<sup>TM</sup>  
SURFACE

Clinell's first plastic-free  
medical wipe



Environmentally friendly without compromising  
efficacy, quality and compliance



Wipes derived  
from sustainable  
sources



Effective against  
VRE, norovirus and  
*Candida auris*



Up to 2.4m<sup>2</sup>  
coverage with  
each wipe

# Clinell Plastic-free Universal Wipes

## Balancing sustainability and infection prevention objectives are challenging.

The UK strives towards a net zero healthcare system, there is a desire to minimise medical consumables. The environment plays a significant role in the transmission of infection and healthcare workers are already at capacity. Providing simple and trusted solutions to improve cleaning compliance is essential.

Clinell is loved and trusted by healthcare workers all over the world. Now you can minimise plastic and achieve sustainability objectives without compromising on efficacy or patient safety, with Clinell Plastic-free Universal Wipes.

### Gentle on planet, tough on microbes



The same trusted Clinell Universal formula, they clean and disinfect in one easy step. Effective against MRSA, VRE, coronavirus, norovirus and more in 60 seconds.

### Improve cleaning compliance



GAMA Healthcare's education platforms and wall mounted dispensers support healthcare workers to decontaminate the environment regularly, protecting their patients and residents from infection.

### Do more, waste less



Clinell Plastic-free Wipes go even further. Each wipe will cover up to 2.4m<sup>2</sup>, which means that a single wipe is enough to cover a full bed mattress.

Product	Unit of issue	Product code
Clinell Plastic-free Universal Wipes	Pack of 140 wipes	CW140PF

# Efficacy data

Bacteria example	Test	Log reduction	Contact time	Condition
<i>Acinetobacter baumannii</i>	EN 13727	5.52	10 seconds	Dirty + Erythrocytes
	EN 16615	5.98	2 minutes	Dirty + Erythrocytes
<i>Enterococcus faecalis</i>	EN 13727	5.42	10 seconds	Dirty + Erythrocytes
	EN 16615	5.77	2 minutes	Dirty + Erythrocytes
<i>Enterococcus hirae</i>	EN 13697	5.01	2 minutes	Dirty + Erythrocytes
	EN 13727	5.00	10 seconds	Dirty + Erythrocytes
	EN 16615	5.74	10 seconds	Dirty + Erythrocytes
<i>Escherichia coli</i> K12	EN 13727	5.08	10 seconds	Dirty + Erythrocytes
<i>Klebsiella pneumoniae</i>	EN 13727	5.48	10 seconds	Dirty + Erythrocytes
Methicillin-resistant <i>Staphylococcus aureus</i>	EN 13727	5.07	10 seconds	Dirty + Erythrocytes
	EN 16615	6.03	2 minutes	Dirty + Erythrocytes
<i>Pseudomonas aeruginosa</i>	EN 13697	5.16	2 minutes	Dirty + Erythrocytes
	EN 13727	5.00	10 seconds	Dirty + Erythrocytes
	EN 16615	6.20	10 seconds	Dirty + Erythrocytes
Vancomycin resistant <i>Enterococci</i> (VRE)	EN 13727	6.00	1 minute	Dirty + Erythrocytes
	EN 16615	5.97	2 minutes	Dirty + Erythrocytes

Mycobacteria example	Test	Log reduction	Contact time	Condition
<i>Mycobacterium bovis</i>	EN 14348	6.10	2 minutes	Dirty + Erythrocytes

Yeast example	Test	Log reduction	Contact time	Condition
<i>Candida albicans</i>	EN 13624	4.00	1 minute	Dirty + Erythrocytes
	EN 16615	5.45	10 seconds	Dirty + Erythrocytes

Virus example	Test	Log reduction	Contact time	Condition
<b>Adenovirus</b>	EN 14476	5.82	1 minute	Dirty + Erythrocytes
<b>Hepatitis B</b>	ASTM E1052	3.75	1 minute	Clean
<b>Hepatitis C</b>	EN 14476	5.67	1 minute	Dirty + Erythrocytes
<b>Herpes simplex type 1</b>	ASTM E1053	5.08	2 minutes	Dirty
<b>HIV</b>	EN 14476	4.33	30 seconds	Dirty + Erythrocytes
<b>MERS-CoV</b>	EN 14476	4.22	1 minute	Dirty + Erythrocytes
<b>Norovirus</b>	EN 14476	4.00	1 minute	Dirty + Erythrocytes
<b>Rotavirus</b>	EN 14476	4.04	1 minute	Dirty + Erythrocytes
<b>SARS-CoV-2</b>	EN 14476	4.53	30 seconds	Dirty + Erythrocytes
<b>Vaccinia virus</b>	EN 14476	4.00	15 seconds	Dirty + Erythrocytes
	EN 16615	4.38	1 minute	Dirty + Erythrocytes
	EN 16777	5.74	15 seconds	Dirty + Erythrocytes

Always read the label and product information before use.

GAMA Healthcare Ltd.

The Maylands Building, Maylands Avenue, Hemel Hempstead, Hertfordshire, HP2 7TG, UK.  
+44 (0)20 7993 0030 | info@gamahealthcare.com | www.gamahealthcare.com

JBN250157

**gama**  
healthcare